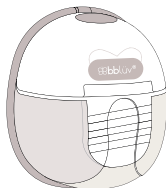
# MümFlow

- EN Heating Wearable Breast Pumps
- FR Tire-laits nomade et chauffant
- ES Sacaleches térmicos portátiles
- IT Riscaldamento dei tiralatte indossabili
- DE Erwärmung tragbarer Milchpumpen
- PT Aquecimento das bombas tira leite vestíveis

MÜM FLOW - B0212 - B0215 - S18

*Please read the instruction manual carefully before use.*

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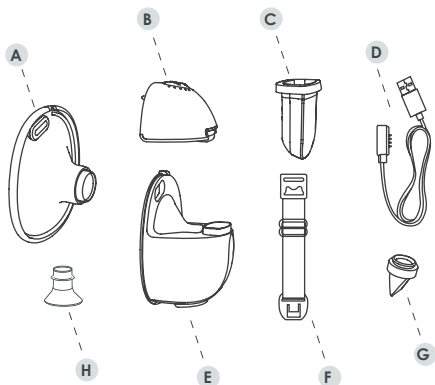
**G**

Valve\*  
 Valve\*  
 Valva\*  
 Valva\*  
 Valva\*  
 Valva\*

**F**

Bra extension strap  
 Sangle d'extension de  
 soutien-gorge  
 Correa de extensión del  
 sujetador  
 Cinghia di estensione del  
 reggiseno  
 BH-Verlängerungsband  
 Faixa de extensão do sutiã

**H**

Flanges breast shield\*\*  
 Colerette de téterelles\*\*  
 Bidas escudo mamario\*\*  
 Flange scudo mammario\*\*  
 Flansche Brustschild\*\*  
 Flanges do resguardo do  
 peito\*\*

\* Recommendation: change every 3 months  
 Recommandation : à changer tous les 3 mois  
 Recomendación: cambiar cada 3 meses  
 Raccomandazione: cambiare ogni 3 mesi  
 Empfehlung: alle 3 Monate wechseln  
 Recomendação: mudar de 3 em 3 meses

\*\* Recommendation: change every 6 months  
 Recommandation : à changer tous les 6 mois  
 Recomendación: cambiar cada 6 meses  
 Raccomandazione: cambiare ogni 6 mesi  
 Empfehlung: alle 6 Monate wechseln  
 Recomendação: mudar de 6 em 6 meses

— — — EN

FR

ES

DE

IT

PT

MümFlow

**Mumflöw**

Model : B0212 B0215

Heating Wearable Breast Pump

Tire-lait portable et chauffants

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This device conforms to the provisions of the EU 2017/745 Medical Device Regulation (MDR).

The following standards apply to the design and manufacture of the device.

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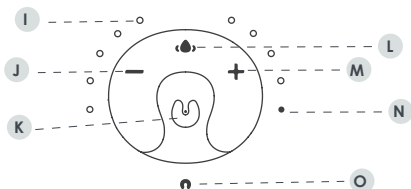
|                                                     |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                              |
|-----------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <p><b>I</b> Indicator lights</p>                    | <p>The device offers 9 suction levels, allowing you to customize the process to suit your needs. The lights indicate the current suction level, with 1 light representing the lowest suction level and 9 lights indicating the highest suction level available.</p>                                                                                                                                                                                                                          |
| <p><b>J</b> Decrease level button</p>               | <p>Press Decrease level to shift to a lower level than the current selection. (1 through 9)</p>                                                                                                                                                                                                                                                                                                                                                                                              |
| <p><b>K</b> Power button/ mode selection button</p> | <p>Press and hold to turn the device on or shut it. Press again to select the desired mode. <br/> <b>Massage:</b> The indicator light flashes when in massage mode. This mode is used to help stimulate a letdown; it has 9 different suction levels. <br/> <b>Expression:</b> The indicator light is stable when in expression mode. This mode is generally used after massage mode, once the milk starts flowing. This mode is slower but stronger; it has 9 different suction levels.</p> |

|                                       |                                                                                                                                                                                                                                                                                                                                      |
|---------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>L</b> Hot compress button          | Press to turn the heating function on or shut it.                                                                                                                                                                                                                                                                                    |
| <b>M</b> Increase level button        | Press Increase level to shift to a higher level than the current selection. (1 through 9)                                                                                                                                                                                                                                            |
| <b>N</b> Hot compress display         | When the heating function is on, the indicator light will flash showing it is heating. When the light stabilizes, it indicates the pump is in "keep warm" mode, the temperature will then stop fluctuating                                                                                                                           |
| <b>O</b> Power display/ battery state | During the use of the breast pump, when the power is sufficient, the power display is green. When the power is low, the light will turn red. During charging, the power display will flash red. As the battery nears full charge, the light will flash green. Once the battery is fully charged, the green light will remain steady. |


## ASSEMBLING YOUR MUMFLÖW

Before first use, please make sure to thoroughly clean your breast pump. You can refer to the **Cleaning** section of this manual (p.#) for more information on how to clean it properly. Make sure to clean your hands as well before you proceed with the assembly.


1. Attach the silicone valve **G** to the milk collector **E**, make sure the valve is installed horizontally.

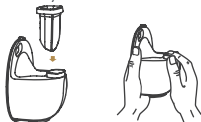
2. Put the silicone diaphragm **C** into the milk collector **E** and press it with your hands to ensure a perfect seal.

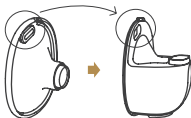3. To install the breast shield **A** on milk collector **E**, make sure to align the grooves correctly. Make sure the magnetic contact on the breast shield **A** is aligned with the one on the milk collector **E**, and then press the edges with both hands to ensure the seal complete.

4. Attach the pump motor **B** to the milk collector **E**, align the notch under the pump with the silicone diaphragm **C**, and press it downward to ensure a complete seal.
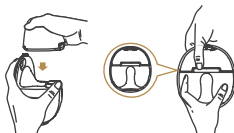
### SELECT THE SIZE OF YOUR BREAST SHIELD

Four different sizes are possible with your Mumflöw: three sizes with flange breast shield 17mm, 19mm, 21mm and the breast shield alone 24 mm.

Making sure you use the right size will maximize your milk expression and make your experience more comfortable. Expressing milk should not be painful in any way.

To help determine the best size for you, measure the diameter of your two nipples as they may differ in size. It is advisable to measure the nipple at rest and not just after feeding. Try to measure across the middle of the nipple and do not include the areola in your measurements. If your result is:

- 15 millimeters or less, opt for the 17 mm flange breast shield;
- between 15 and 17 mm, opt for the 19 mm flange breast shield
- between 17 and 19 mm, use the 21 mm flange breast shield;
- between 19 and 22 mm, opt for a 24 mm breast shield

Refer to the drawings below for a reference of what your nipple should look like while pumping.

Please see below for signs indicating you should consider changing the size of your flange. If you experience any of these situations or are still unsure about any part of the process, do not hesitate to contact a lactation consultant or a breastfeeding specialist.

- Your nipple rubs along the side of the tunnel to the point of causing discomfort or redness
- Your nipple and much of your areola are being pulled into the tunnel
- Your nipple or areola turns white while you express milk
- You do not feel that all the breast milk has been expressed after your pumping session

## HOW TO USE YOUR MUMFLÖW

**1.** Wash your hands thoroughly and assemble your Mumflöw according to the illustrated sequence in this manual. While doing so, make sure every component is properly sanitized and free of any visible damage.

**2.** Make sure the pump motor **B** battery is sufficiently charged and that the power display **O** is green.

**3.** Press the breast shield **A** firmly against the breast, leave no gaps and make sure your nipple is centered. Keep the pump upright. Be careful not to tilt the breast pump and the chest, this could affect the suction. For

more information on the correct fit and size for you, please consult the Breast Shield size section of this manual

4. Press and hold the power button **K** to start the breast pump.

5. Press the power button **K** again to switch modes. There are 2 modes for you to choose from: expression mode and massage mode. The indicator light **N** flashes when in massage mode and is always on when in expression mode.

6. Each mode features 9 suction levels, each represented by a corresponding display **I**. When all lights are illuminated, it shows you are at the highest suction level, level 9. As the level increases, the lights will turn on from left to right. You can adjust the suction level by pressing the level up **M** and level down **J** buttons.

7. Press the hot compress button **L** to turn the heating function on and off. When the heating function is on, the

indicator light **N** will flash showing it is heating. When the light stabilizes, it indicates the pump is in "keep warm" mode, the temperature will then stop fluctuating. If the hot compress indicator light **N** does not light up, please check whether the magnetic contact of the breast shield **A** is in good contact with the pump motor **B**.

**8.** To turn off the heating function, press the hot compress button **L**, if the display **N** does not light up, it means the heating function is off.

**9.** Press and hold the power button **K** to turn off the breast pump.

## NOTES

- A regular pumping session can last from 15 to 30 minutes it is different with each mom. The device will shut off automatically after 20 minutes of use. If you feel you need to pump more, feel free to start the breast pump again.

- This model features a built-in memory function, which automatically resumes the previous mode and suction level when you turn the machine back on.

- The level of milk in the milk collector **E** should not exceed the maximum capacity (180 ml). Please empty the milk collector after each breast to prevent overflowing.

- To prevent backflow and maximize milk expression, make sure to remain in a comfortable seated position throughout the session. Make sure to keep the breast pump upright.

### ADJUSTING THE BRA EXTENTION

1. To use this extension, ensure you are wearing a nursing bra or shirt with a detachable clip.
2. Unclip your nursing bra and attach the extension **F** to the upper clip of your bra. Make sure the adjusting loop faces upward for easier adjustment later. Loosen the strap as needed.
3. Follow the instructions in the "How to use your Mumflöw" section to install the breast pump.
4. Attach the second end of the extension **F** to the remaining clip and tighten the strap until the bra or shirt comfortably covers the breast pump.

Picture 1


Picture 2

Picture 3

### HOW TO REMOVE YOUR MUMFLÖW

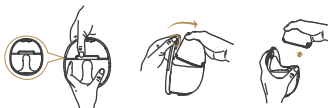1. When you are done with your session, press and hold the power button **K** to turn off the device.
2. Use your finger to break the seal between the breast and the breast shield **A** before carefully removing the pump.
3. Separate the pump motor **B** from the milk collector **E** by tilting the motor forward and separating the magnetic contact first.

4. Align the spout on the milk collector **E** with the opening of a sterilized milk storage container and pour the milk in.

## STORING BREAST MILK

1. Before storing your milk, make sure to mark the date on the container. The storage life in a refrigerator is up to 4 days<sup>1</sup> and 6 to 12 months<sup>1</sup> in a freezer. Do not use breast milk that has been stored longer than recommended or if you have any doubts of its period of storage.
2. Thaw breast milk by placing it in the refrigerator for a few hours
3. Use thawed breast milk within 24 hours and never put it back in the freezer<sup>1</sup>.
4. Heat breast milk under warm water and gently swirl the container to mix the cream and the milk back together. Do not shake the milk as this could break down some of its nutrients<sup>2</sup>. You can also warm the milk by placing the container in a pan of warm water or a milk heater. Do not warm directly on the stove or in a microwave oven.
5. Always test the milk's temperature before feeding it to

a baby. Place a few drops on your inner wrist. It should feel lukewarm, close to your own body temperature.

1. Centers for Disease Control and Prevention, Proper Storage and Preparation of Breast Milk, [Internet], Atlanta, GA, USA: US Department of Health and Human Services; 2018 [updated 2019 June; cited 2021 July] Available at: [https://www.cdc.gov/breastfeeding/recommendations/handling\\_breastmilk.htm](https://www.cdc.gov/breastfeeding/recommendations/handling_breastmilk.htm)
2. Office of Women's Health, Pumping and storing breastmilk, [Internet], Silver Spring, MD, USA: US Department of Health and Human Services; 2018 [updated 2018 Jan; cited 2021 July] Available at: [www.womenshealth.gov/breastfeeding/pumping-and-storing-breastmilk](http://www.womenshealth.gov/breastfeeding/pumping-and-storing-breastmilk)

## BATTERY

1. During the use of the breast pump, when the power is sufficient, the power display  is green. When the power is low, the light will turn red. Make sure to charge it in time.

2. The Mumflöw cannot operate while the battery is charging. During charging, the power display  will flash red. As the battery nears full charge, the light will flash green, and once fully charged, the green light will remain steady.

Charging time : about 2 hours

Time of breast pumping usage with heating function :

About 80 min

Time of breast pumping usage without heating function :

About 180 min

Auto shut off : 20 min

## CLEANING AND STERILISING YOUR MUMFLÖW

Make sure to clean your Mumflöw before first use and after each use.

1. Except for the pump motor , rinse all parts in cold water to remove any breast milk residue or dust.
2. Makes sure to disassemble all parts following the illustrated sequence in this manual.
3. Sterilise all parts that come in contact with breast milk\* in a clean pot with boiling water or any other preferred

sterilisation method. Make sure that each element has enough room to move freely and to avoid direct contact with the bottom by placing the elements on a rack or a steamer

**4.** After 5 minutes in boiling water, carefully remove the individual parts with tongs and let air dry completely on a clean paper towel or cloth.

**5.** Make sure to clean your hands thoroughly before putting the parts back together.

**6.** Do not wet or immerse the pump motor in water or other liquid. Clean it with a clean and dry cloth.

\*Breast shield **A**, Silicone Diaphragm **C**, Milk Collector **E**, Valve **G**.

## NOTES

- Parts may warp or have a shorter lifespan if they directly touch the bottom of the pan during the boiling (sterilization) process.
- Be careful when removing and cleaning the valve, if the valve is damaged, the breast pump will not work properly.
- To clean the valve, please rub it gently in warm water with a few drops of cleaning liquid. Do not insert any objects into the valve, which may cause damage.

## WARNINGS

- Make sure to clean the pump as instructed in this user manual.
- Make sure to assemble the pump correctly as instructed in this user manual.
- Wash your hands thoroughly before assembling clean components.
- To avoid leakage, remove any residue around the rim before assembling.
- The breast pump does not work when charging.
- Check whether the local power voltage matches the

device's specifications before connecting it to a power source.

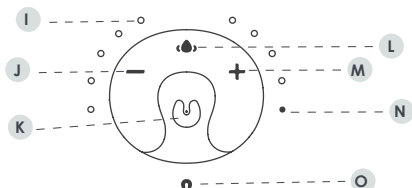
- Do not modify the device without the manufacturer's authorization. This could cause improper functioning.
- If the DC adapter is damaged, it should be repaired by a professional.
- If the device is not used for a long time, the battery will fully discharge.
- The pump motor has a built-in lithium. Remove it from the device before discarding it. Do not dispose of this product with ordinary household waste. Look up the proper procedure for disposing of electronics in your region.
- The lithium battery may only be replaced by an authorized service provider. There is a risk of explosion if the battery is replaced with an improper model.
- Never immerse the pump motor unit in water or clean it with a wet sponge or cloth.
- Keep all components out of the reach of children. Children should be supervised to ensure that they do not play with the device. Keep the Mumflöw out of reach of children to avoid risk of strangulation from cables or the risk of swallowing small parts.
- This device is not intended for use by children. The Mumflöw should always be operated by a responsible adult.
- Please store in a cool and dry place.
- Flange with magnetic charge can be cleaned under water, but It needs to be dry when using.

| TECHNICAL SPECIFICATIONS |                 |
|--------------------------|-----------------|
| Name and Model           | Mumflöw – B0212 |
| Power                    | 3,7W            |
| Voltage                  | 5V              |
| Current                  | 2A              |

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:  
(1) This device may not cause harmful interference, and  
(2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

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