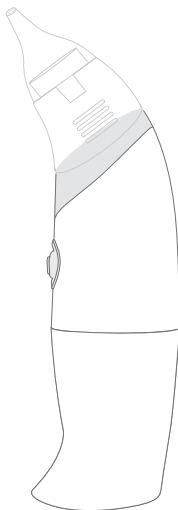
Rinö

- EN Filter-Free Nasal Aspirator
- FR Aspirateur nasal sans filtre
- ES Aspirador nasal sin filtros
- IT Aspiratore nasale senza filtro
- DE Filterloser Nasenaspirator
- PT Aspirador Nasal sem filtro

B0127 - Rinö - 243 - instruction manual - V1

Please read the instruction manual carefully before use.

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APPLIED STANDARDS:

This device conforms to the provisions of the EU 2017/745 Medical Device Regulation (MDR).

The following standards apply to the design and manufacture of the device:

- IEC/EN 60601-1 Medical Electrical Equipment – Part 1: General requirements for basic safety and essential performance
- IEC/EN 60601-1-2 Medical electrical equipment – Part 1-2: General requirements for basic safety and essential performance – Collateral Standard: Electromagnetic compatibility – Requirements and tests
- IEC/EN 60601-1-6 Medical electrical equipment – Part 1-6: General requirements for safety – Collateral standard: Usability
- IEC/EN 60601-1-11 Medical electrical equipment – Part 1-11: General requirements for basic safety and essential performance – Collateral Standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment
- EN ISO 10993-5 Biological evaluation of medical devices - Part 5: Tests for in vitro cytotoxicity
- EN ISO 10993-10 Biological evaluation of medical devices - Part 10: Tests for skin sensitization
- EN ISO 10993-23 Biological evaluation of medical devices - Part 23: Tests for irritation


Asia Connection Co., Ltd. 16 F-1, NO. 16, Jian Ba Rd.,
Chung Ho District, New Taipei City 235603, Taiwan

Advena Ltd. Tower Business Centre, 2nd Flr, Tower Street, Swatar, BKR 4013, Malta

	Caution/ Avertissement/ Atención Attenzione/ Vorsicht/ Atencão		Manufacturer/ Fabricant/ Fabricante Produttore/ Hersteller/ Fabricante
	Importer/ Importateur/ Importador Importatore/ Importeur/ Importador		Medical device/ Appareil médical Dispositivo médico/ Dispositivo medico Medizinisches Gerät/ Dispositivo médico
	Refer to instruction manual Reportez-vous au manuel d'instructions Consulte el manual de instrucciones Consultare il manuale di istruzioni Siehe Gebrauchsanweisung Consultar o manual de instruções		IEC 60417-5333, Type BF applied part. IEC 60417-5333, Pièce appliquée de type B. IEC 60417-5333, parte de contacto tipo BF. IEC 60417-5333, Parte applicata di tipo BF. IEC 60417-5333, Angewandtes Teil Typ BF. IEC 60417-5333, peça aplicada Tipo BF.
	Reference number/ Numéro de référence Número de referencia/ Numero di riferimento Referenznummer/ Número de referência		CE marking of conformity Marquage CE de conformité Marcado CE de conformidad Marchio di conformità CE CE-Kennzeichnung der Konformität Marcação CE de conformidade
	Batch code, see the LOT at the bottom of the device Code de lot, voir le LOT au bas de l'appareil Código de lote, véase el LOT en la parte inferior del aparato Codice del lotto, vedere il LOT sul fondo del dispositivo Chargencode, siehe LOT auf der Unterseite des Geräts Código do lote, ver o LOT na parte inferior do dispositivo		Printed in paper (recycling code /22 PAP) Imprimé sur papier (code de recyclage /22 PAP) Impreso en papel (código de reciclaje /22 PAP) Stampato in carta (codice di riciclaggio /22 PAP) Bedruckt in Papier (Recycling-Code /22 PAP) Impresso em papel (código de reciclagem /22 PAP)
	Date of manufacture (YYMM). See the date or manufacture at the bottom of the device Date de fabrication (YYMM). Voir la date de fabrication au bas de l'appareil Fecha de fabricación (AA/MM). Véase la fecha de fabricación en la parte inferior del dispositivo Data di fabbricazione (YYMM). Vedere la data di fabbricazione sul fondo del dispositivo Datum der Herstellung (JJMM). Siehe das Herstellungsdatum auf der Unterseite des Geräts Data de fabrico (YYMM). Ver a data de fabrico na parte inferior do dispositivo		Authorized representative in the European Community/ European Union Représentant autorisé dans la Communauté européenne/Union européenne Representante autorizado en la Comunidad Europea/ Unión Europea Rappresentante autorizzato nella Comunità Europea/ Unione Europea Bevollmächtigter Vertreter in der Europäischen Gemeinschaft/Europäischen Union Representante autorizado na Comunidade Europeia/ União Europeia
IP22	Protected from the touch of fingers and objects greater than 12 millimeters. Protégé du toucher des doigts et des objets de plus de 12 millimètres. Protegido del toque de dedos y objetos superiores a 12 milímetros. Protetto dal tocco di dita e oggetti superiori a 12 millimetri. Geschützt vor Berührung mit den Fingern und Gegenständen größer als 12 Millimeter. Não tocar com os dedos ou objetos maiores que 12 milímetros.		This device may be used multiple times on a single patient Cet appareil peut être utilisé plusieurs fois sur un même patient Este dispositivo puede utilizarse varias veces en un mismo paciente Questo dispositivo può essere utilizzato più volte su un singolo paziente Dieses Produkt kann mehrfach an einem Patienten verwendet werden Este dispositivo pode ser utilizado várias vezes num único doente
	Unique device identifier Identifiant unique du dispositif Identificador único del dispositivo Identificatore univoco del dispositivo Eindeutige Gerätekennzeichnung Identificador único do dispositivo		
	Do not dispose of this product with ordinary household waste. Ne jetez pas ce produit avec les ordures ménagères ordinaires. No elimine este producto con los desperdicios habituales del hogar. Non smaltire il prodotto insieme agli altri rifiuti domestici. Entsorgen Sie dieses Produkt nicht im normalen Hausmüll. Não elimine este produto junto com resíduos domésticos normais.		

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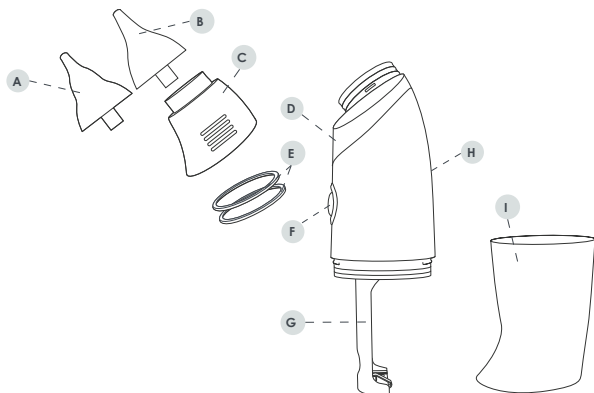
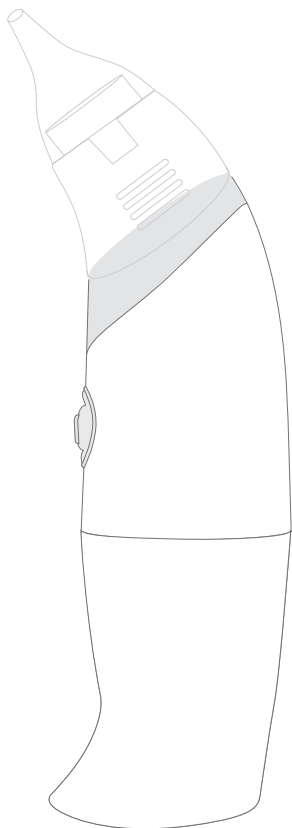
BEFORE USE

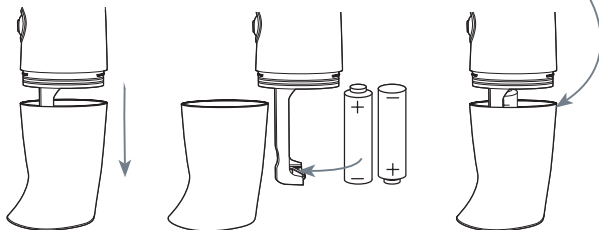1. Carefully but firmly remove the lower cover **I** by pulling it straight down. Do not twist, bend, or force the cover open.
2. Insert two AA batteries (not included) in the battery compartment, **G** making sure to align them with the polarity markings.
3. Make sure that the two silicone rings are properly placed on the grooves. This will ensure the product is sealed.

4. Replace the lower cover **I** and push it towards the suction tube **H** until both pieces click back in place.

NOTE:

Keep batteries out of reach of children and dispose of them properly. Do not mix different types of batteries, or use new ones with old ones

HOW TO USE YOUR RINÖ

1. Choose the appropriate silicone tip **A** / **B** based on the size and shape of your baby's nostril, as well as the degree of congestion. Make sure it is securely attached to the mucus collection cup **C**.
2. Hold your baby in an upright position and gently insert the silicone tip **A** / **B** into your baby's nose. To avoid risk of injury, do not insert the tip too deeply, press against the wall of the nostril, or completely block the nostril. For best results, the tip should be pointed towards the outer side of the nostril at a 90-degree angle.
3. Press and hold the Power button **F** to start the gentle and steady suction process. A few seconds of use should be enough. Release the Power button **F** and remove the device from your baby's nostril when done.

NOTE:

(1) It may take a bit of time for mucus to be visible in the collection cup. However, the mucus will be visible as soon as you detach the collection cup for cleaning.

(2) Do not use the Rinö if the level of mucus is about to exceed the opening channel (about 4 ml).

INTENDED USE

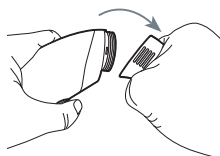
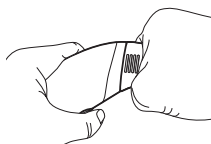
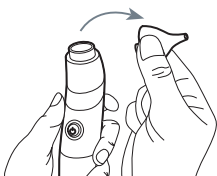
The Rinö provides intermittent suction for removing nasal secretions and mucus to alleviate nasal discomfort.

CLEANING & MAINTENANCE



For reasons of hygiene, it is important to wash your Rinö before and after each use. Please carefully follow the instructions.

Quick cleaning your Rinö

1. Make sure that the lower cover **I** has been firmly closed and locked in place.
2. Remove the silicone tip **A** / **B**.
3. Hold the mucus collection cup **C** between your thumb and your index finger.
4. Gently bend the mucus collection cup **C** downwards to detach it from the suction tube **H**.

5. Every few times you use the device, detach the silicone rings **E** from the collar to clean them.
6. Thoroughly brush and rinse the inside and outside of all the parts. Ensure there are no residues remaining.
7. Boil a pot with clean water. Put the silicone tip **A** / **B** and rings **E**, and the mucus cup **C** in the water and stir them for 3 to 5 minutes. The parts can also be sterilised in steam cleaners, or with a UV sterilizer (such as bblüv Üvi). Never use your bare hands to take out the parts out of the boiling water to avoid burns.
8. Thoroughly wash the collar, the pillar inside as well as the air vent with warm soapy water before rinsing under tap.
9. Make sure all parts are dry before reassembling the device. When replacing the mucus collection cup **C**, make sure it is properly aligned with the positioning hole.

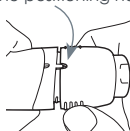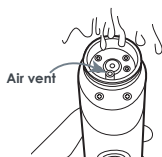
NOTE: When using the Rinö, if the accumulated volume of mucus exceeds the safety limit or if mucus drips out the air vent on the back of the collar, please deep clean your device.


Deep cleaning your Rinö

10. Follow steps 1 through 5 of the Quick Cleaning section of this manual.

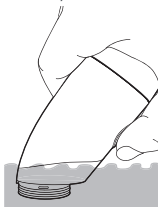11. Immerse the collar of the suction tube **D** in clean water, as shown below. Press and hold the Power button **F** for 15 seconds to clean the inner pump. Change the water and repeat if necessary.

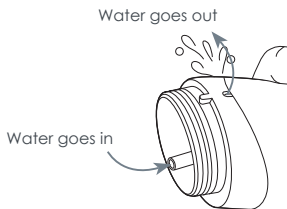12. Water flows in and out of the inner pump, as shown below.

13. Remove the Rinö from the water once you finish cleaning. Press and hold the Power button **F** for 30 seconds to allow the inner pump to dry.

14. Dry the collar of the suction tube **D** and reassemble the parts.

WARNINGS

- The Rinö is not a toy. Keep out of reach of children to prevent accidental swallowing of small parts or accessories.
- The Rinö should always be used by an adult or a responsible person.
- The Rinö is not a medical device and is designed to be used solely for the suction of nasal mucus. This device is not intended for use in the mouth or in ears.
- Do not use in case of wounds, inflammation, or bleeding from the nose.
- The Rinö is intended for personal, not commercial use.
- Before use, make sure the silicone tip is firmly in place on the mucus collection cup.
- If you do not intend to use the product for a long period of time, remove the battery.
- In case of infection, fever, nasal irritation, or hemorrhage, immediately stop use and consult your family doctor.
- Do not use the Rinö if your baby's nose is showing signs of injury, cuts, irritation or inflammation.

- Counter-indications: Haemophilia, dilated vessels of the mucous membrane of the nasal septum, frequent nosebleeds, medical treatment with anticoagulants, congenital malformations (e.g. Palatal fissure).
- The Rinö includes electrical parts which required protection when cleaning. Please follow the cleaning instructions provided in the **Cleaning and Maintenance** section of this manual.

TECHNICAL PARAMETERS

Recommended battery	2 x 1,5V alkaline batteries; type AA
Suction Strength	53 ~ 65 kPa
Weight	160g ± 15g (5,6oz ± 0,5oz)
Size	65 x 40 x 194 mm (2,5 x 1,5 x 7,6 in)
Waterproof Level	IP22
Duty Cycle	30 sec. ON/ 2 min. OFF

OPERATING CONDITIONS

Temperature	5°C to 40°C (21°F to 104°F)
Humidity	15% to 90% RH
Atmospheric Pressure	70kPa to 106kPa

STORAGE AND SHIPPING CONDITIONS

Temperature	-13°F to 158°F (-25°C to 70°C)
Humidity	≤ 90% RH, non-condensing
Atmospheric Pressure	70kPa to 106kPa

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